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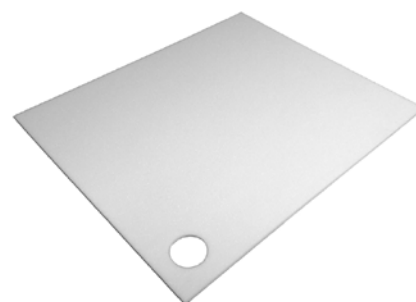
Codice	Misure (cm)
03.04.8040	45 x 38

**Destinazione d'uso**

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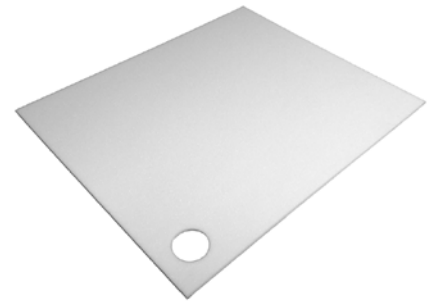
**Use**

The Transfer Systems are intended for facilitating the manual handling operations of patients through thrust, traction, sliding maneuvers. They have been designed for use in hospitals, nursing homes and healthcare facilities and for home care.

**Description**

The NON SLIP cushion is used along with the transfer system MIDISLIDE for the correct positioning of the patient in the bed. It is placed under the feet when the patient is lying with his legs bent, then, with a slight push of legs, the patient will slide towards the headboard thanks to the transfer system.

Materials	
Non Slip	High Density Foam
Maintenance and Cleaning	
Disinfection is possible with 5% diluted disinfectants even with alcohol base	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr.Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
CND Code	Y123099
RDM Code	1167517/R