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Codice	Misure (cm)
05.00.AP903	71 x 12 x 0.8

**Categoria**

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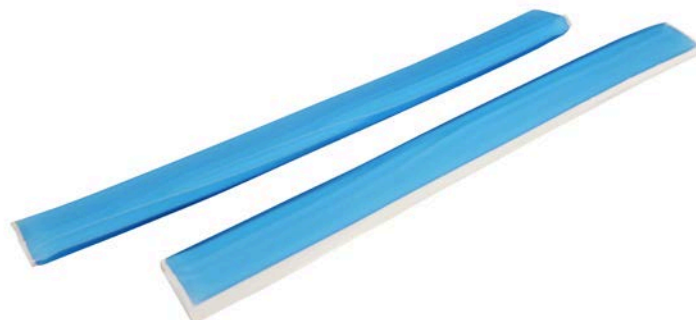
### Category

Gel Positioner for Operating Table.

### Description

It is designed for patients in lithotomy position, urology or gynecology procedures. These pads protect patients from pressure insults or shear that could occur while coming in contact with metal stirrups. They need not be removed after each procedure. This eliminates the need for tedious wrapping and rewrapping and provides superior pressure and shear protection. The ANKLE STRAP STIRRUP PAD are a suggested complement to our Candy Cane Stirrup Pads.

### Materials

It is made of an amorphous polyurethane gel with a density of 0.99 gr/cm<sup>3</sup>.

### Maintenance and Cleaning

- The gel positioners can be easily cleaned with any cleaning agent and / or disinfectant for the operating room.
- Avoid immersing the gel positioners in the detergent and / or disinfectant for a long time.

### Storage and Disposal

- Keep the positioners, if possible, on a level surface.
- Avoid placing heavy or sharp objects on the positioners.
- Keep the gel temperature between 50 ° and -18 °.
- Keep away from sources of light, moisture and heat.
- Dispose of the device in compliance with current regulations on environmental protection and separate waste collection.

### Package

2 Pieces.

### Warranty

24 months for manufacturing defects with the exception of consumable parts.

### ISO Code

04.33.09

### CND Code

Y033399

### RDM Code

90736/R

### Medical Device Class

I complies with Directive 93/42/CEE