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Product Name
Pool Lift Balneo with Seat.

Code	Capacity (Kg)	Article
01.01.BALNEO	200	Lift
01.01.BALNEOSEAT	200	Seat

Use
The lift is used for immersion in the swimming pool disabled patients that need hydrokinesitherapy.

Description

The BALNEO is a passive lift and it allows patients with important functional limitations to enter or come out from the swimming pool. It is equipped with a removable LINAK control unit placed on the mast, hand remote control, 4 anchor points sling bar with safety system, handle for moving, base with 4 screws to fix the lift to the edge of the swimming pool, an emergency stop red button that blocks the flow of electricity, a mechanical safety lowering in case of black out and an additional electric safety lowering system in case of malfunction of the remote control. The boom descends to the level of the pool and rotates 360°. The supplied seat (code 01.01.BALNEOSEAT) is fixed to the lift frame by a support arm, which allows a comfortable and safe patient transfer.

Code	Base Dimensions	Minimum Height at Sling Bar	Maximum Height at Sling Bar	Sling Bar Length	Sling Bar Width	Floor to Handle Height	Handle Width	Rotation Range	Dimensions Folded	Weight Capacity	Weight	Carton Size
Measures in Cm												
01.01.BALNEO	32x32	31	165	56	67	112	67	129	139x66x50	200	49	\\

Electrical Group	
LINAK	24 V
Seat 01.01.BALNEOSEAT Dimensions	
Seat Width	40 cm
Seat Depth	38 cm
Support Arm Length	140 cm
Maximum Seat Height from the Edge of the Pool	84 cm
Minimum Seat Height from the Edge of the Pool	-80 cm
Lift Materials	
Frame	Steel
Seat Materials	
Seat	PVC
Support Arm	Steel
Maintenance and Cleaning	
Cleaning can be done with a damp cloth and neutral soap after making sure the battery is not charging	
Storage and Disposal	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
ISO Code DM332/99	12.36.03.006
ISO Code DPCM 12.01.2017	12.36.03.006

CND Code	Y123603
----------	---------