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### Use

Polymoove Positioners are a valid aid for healthcare professionals for patient positioning. They are designed to give comfort, support, stability and protection of vulnerable areas. The positioners adapt to the morphology of the body in an excellent way thanks to the composition of granulated foam spheres. The various shapes are easily combined with each other and ideal for an efficient anti-decubitus prophylaxis. The rectangular positioner is used as a pillow, as a stabilizer for lower and upper limbs and for support in moving the patient in a lateral position.

### Description

The RECTANGULAR PROTECTION POLYMOOVE is internally composed of granulated foam contained in a protective cotton casing.

The Polymoove filling material offers optimal patient support by helping to distribute body weight in a lying or sitting position. The cushion is covered with an external cover in Obatex fabric, removable thanks to the zip on the side. The fire retardant, breathable, bi-elastic, waterproof and antibacterial OBATEX lining is made of polyurethane and polyester, sterilizable in an autoclave.

The Polymoove does not deform after being washed and dried in the washing machine, the positioning cushions can be used for many cycles without altering their shape. Their low moisture absorption rate facilitates cleaning and environmental sustainability.

The use of positioners with the Evopore plate is recommended for kinesiology practices.

### Positioning Kit:

- POLYMOOVE Rectangular Positioner 15x40 cm (ref. 03.05.POL10)
- Rectangular sheet in EvoPore POLYMOOVE 50x50x3 cm (ref. 03.05. R4018)
- POLYMOOVE Rectangular Positioner 50x75 cm (ref. 03.05.POL32)
- POLYMOOVE Rectangular Positioner 55x95 cm (ref. 03.05.POL34)
- POLYMOOVE Linear Dorsal Positioner 220x36 cm (ref. 03.05.POL91)

Materials	
Padding	Espanso Granulato in Polipropilene (EPP)
Protective Overwrap	Cotton
External cover	Obatex
External Cover Maintenance and Cleaning	
	Water temperature not above 95°C or 200°F
	Do not bleach
	Do not iron
	Dryclean with any solvent
	Tumble Dry Max temperature 60°C or 140°F
We recommend washing the cover at least once a year	
Positioner Maintenance and Cleaning	
	Water Temperature not Above 60°C or 140°F
	Do not Bleach
	Maximum Temperature 150°C or 300°F

	Dryclean with any Solvent
	Tumble Dry Medium Heat
We recommend washing the positioner at least once a year	
<b>Storage and Disposal</b>	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
<b>Package and Nr.Items</b>	
Box	1 Piece
<b>Warranty</b>	
European	24 months for manufacturing defects with the exception of consumable parts
<b>Product References</b>	
Manufacturer	OBA AG
Authorized Distributor	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
CND Code	Y060399
RDM Code	225942/R