

**Nome Prodotto**

Talloniera con Lacci.

Codice	Misure (cm)
05.00.AP033	17 x 18 x 1.3

Categoria

Posizionatore in Gel per Sala Operatoria.

Descrizione

La TALLONIERA CON LACCI viene indossata dal paziente che si trova in posizione supina; i lacci di silicone ne permettono la chiusura.

MaterialiLa TALLONIERA CON LACCI è realizzata in gel amorfo di poliuretano con densità di 0,99 gr/cm³ rivestito da un film di poliuretano. I lacci per la chiusura sono in silicone.**Manutenzione e Pulizia**

- I posizionatori in gel posso essere puliti facilmente con qualsiasi detergente e/o disinfettante per la sala operatoria.
- Evitare di immergere per lungo tempo i posizionatori in gel nel detergente e/o disinfettante.

Conservazione e Smaltimento

- Mantenere, se possibile, i posizionatori in una superficie in piano.
- Evitare di appoggiare sui posizionatori oggetti pesanti o taglienti.
- Mantenere la temperatura del gel tra i 50° e i -18°.
- Conservare al riparo da fonti di luce, umidità e calore.
- Smaltire il dispositivo rispettando le normative vigenti in materia di tutela ambientale e raccolta differenziata.

Confezione

2 Pezzi.

Garanzia

24 mesi per difetti di fabbricazione ad esclusione delle parti consumabili.

Codice CND

Y033399

Codice RDM

90736/R

Classe Dispositivo Medico

I conforme alla Direttiva MDR 2017/745



Product Name
Hell Comfort Cup.



Code	Dimensions (cm)
05.00.AP033	17 x 18 x 1.3

Category
Gel Positioner for Operating Table.

Description
It provides comfort and support for heels. May be used to complement anti-embolism stockings. Appropriate for recovery and radiology.

Materials
It is made of an amorphous polyurethane gel with a density of 0.99 gr/cm³. Comes complete with fastening laces.

Maintenance and Cleaning

- The gel positioners can be easily cleaned with any cleaning agent and / or disinfectant for the operating room.
- Avoid immersing the gel positioners in the detergent and / or disinfectant for a long time.

Storage and Disposal

- Keep the positioners, if possible, on a level surface.
- Avoid placing heavy or sharp objects on the positioners.
- Keep the gel temperature between 50 ° and -18 °.
- Keep away from sources of light, moisture and heat.
- Dispose of the device in compliance with current regulations on environmental protection and separate waste collection.

Package
2 Pieces.

Warranty
24 months for manufacturing defects with the exception of consumable parts.

CND Code
Y033399

RDM Code
90736/R

Medical Device Class
I complies with Directive MDR 2017/745

