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Category

Gel Positioner for Operating Table.

Description

It is composed of two pieces:

- Hip pad with hole, designed for use on the fracture table plate, the primary function of these pads are to reduce pressure and shear at the sacral area.
- Post pad, which reduces pressure and shear on the patient's scrotum and upper leg areas. It also eliminates the tedious chore of wrapping and rewrapping the fracture table post with various makeshift items such as gauze, towels, etc. Designed for use on the hip pad, the primary function of these pads are to reduce pressure and shear at the sacral area.

Materials

It is made of an amorphous polyurethane gel with a density of 0.99 gr/cm³.

Maintenance and Cleaning

- The gel positioners can be easily cleaned with any cleaning agent and / or disinfectant for the operating room.
- Avoid immersing the gel positioners in the detergent and / or disinfectant for a long time.

Storage and Disposal

- Keep the positioners, if possible, on a level surface.
- Avoid placing heavy or sharp objects on the positioners.
- Keep the gel temperature between 50 ° and -18 °.
- Keep away from sources of light, moisture and heat.
- Dispose of the device in compliance with current regulations on environmental protection and separate waste collection.

Package

1 Piece.

Warranty

24 months for manufacturing defects with the exception of consumable parts.

CND Code

Y033399

RDM Code

90736/R

Medical Device Class

I complies with Directive MDR 2017/745