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Cuffia Ipotermica Devicap.

Codice	Taglia
06.04.603L	Standard

**Destinazione d'uso**

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### Use

The Devicap cuff is a product designed to perform hypothermal skin treatments. It 'been shown that hypothermia of the scalp is reduced considerably and effective hair loss people undergoing chemotherapy.

### Description

The HYPOTHERMIC DEVICAP consists of a special thermoelastomer which accumulates cold and remains so for a long period of time, variable according to the type of medicine used for the therapy (Doxorubicin, Cyclophosphamide, Cisplatin, Belomycin,% - Fluorouracil, Methotrexate, Vincristine etc.). It is not a single-use device, it can be used several times. Available medical literature.

Materials	
External Coating	Polyester
Internal Coating	Thermoelastomer
Use Precautions	
Devicap must be worn 15 minutes before the treatment and during the same, kept still for about 15 minutes after the end that returns to ambient temperature	
If the therapy exceeds 40 minutes it is advisable to use a second refrigerated cuff when the first loses its effectiveness	
Warnings	
The length of the cuff is individual and depends on the type of therapy and the patient's specific physical conditions that affect the maintenance of the temperature of it	
CAUTION: in some cases, the use of DEVICAP can cause diseases by cooling in those patients with a particular weakening of the physical state	
Maintenance and Cleaning	
Can be washed on soap and water and buffered to dry	
Storage and Disposal	
Keep the cuff in the refrigerator or freezer (up to -18 ° C), inside the given nylon bag	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr.Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
CND Code	T0306
RDM Code	2506354/R