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Compressore Devicare B7.

**Destinazione d'uso**

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Devicare B7 Pump.

## Use

The Devicare B7 pump has to be combined with the mattress Level Top Intensive (code 05.03.TOP-I). The combination of mattress and pump creates a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

| Features  |  |
|---|--|
| Code  | 05.03.B7   |
| Dimensions  | 27 x 20 x 11 cm  |
| Case  | Fire Retardant ABS   |
| Air Mode  | Static and Alternate (A-b-A-b)   |
| Air Output  | 10 LPM   |
| Pressure Range  | 30 to 150 mm Hg  |
| Noise Level   | ≤ 20 Dba   |
| Cycle Time  | 10 min   |
| Power   | 220/240 V-50 Hz  |
| Power Consumption   | 25 Watt  |
| Electric Cable Length   | 4 m, Detachable  |
| Weight  | 2.8 Kg   |
| Fuse  | 1.5 A  |
| Two steel hooks for anchoring to the bed and air filter   |  |
| Visual and sound alarm for low pressure and blackout  |  |
| Warnings  |  |
| The pump can be used only with mattresses recommended by the manufacturer   |  |
| Check the correct assembly of the device with particular attention to the connection of the tubes between the pump and the mattress |  |
| Maintenance and Cleaning  |  |
| Clean the pump with a damp cloth and neutral detergent  |  |
| WARNING: never immerse the compressor in any type of liquid   |  |
| Storage and Disposal  |  |
| Single-user Product   |  |
| Keep away from heat, sparks, open flame or other sources of ignition  |  |
| Dispose of the device in compliance with current regulations on environmental protection and separate waste collection              |  |
| Package and Nr. Items   |  |
| Box   | 1 Piece  |
| Warranty  |  |
| European  | 24 months for manufacturing defects with the exception of consumable parts |
| Product References  |  |
| Manufacturer  | Demarta Virgino S.r.l.   |
| Directive MDR 2017/745  | Class I  |
| ISO Code  | 04.33.06.018   |
| CND Code  | V080701  |
| RDM code  | 665028/R   |

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Fabbricante Demarta Virginio S.r.l.