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2466418/R

### Product Name

Bubble Air Mattress BP4 with Prestige Pump.

### Use

The mattress BP4 has to be combined with Prestige pump (code 05.03.6600A4). The combination of mattress and pump creates a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

Anti-bedsore Kit	
Code	05.03.6600A4+BP4
Mattress Features	
Bubbles Material	Non-Toxic PVC
Number of Bubbles	260
PVC Thickness	300 Microns
Dimensions Inflated	200 x 85 x 16 cm
Dimensions Deflated	320 x 90 x 2 cm
Therapeutic Capacity	150 Kg
Air Mode	Two Ways Dynamic (A-b-A-b)
Flaps 40 cm long at the ends for attachment to the bed	
External cover in fireproof Nylon with press studs	
Cover Maintenance and Cleaning	
🧼 Washing Max 30° C	
Mattress Maintenance and Cleaning	
Clean the mattress with a damp cloth and mild soap	
The mattress has to be dry before use, do not expose it to direct sunlight	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	

Features Pump	
Dimensions	24 x 11.5 x 9 cm
Case	Fire Retardant ABS
Air Mode	Alternate (A-b-A-b)
Air Output	7 LPM
Pressure Range	50 to 120 mm Hg
Noise Level	≤20 DbA
Cycle Time	12 min
Power	220/240 V-50 Hz
Power Consumption	7 Watt
Electric Cable Length	1.7 m
Weight	0.8 Kg
Fuse	1.5 A

Other Features	Two steel hooks for anchoring to the bed
<b>Warnings</b>	
The pump can be used only with mattresses recommended by the manufacturer	
Check the correct assembly of the device with particular attention to the connection of the tubes between the pump and the mattress	
<b>Maintenance and Cleaning</b>	
Clean the pump with a damp cloth and neutral detergent	
WARNING: never immerse the compressor in any type of liquid	
<b>Storage and Disposal</b>	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	

<b>Package and Nr. Items</b>	
Box	1 Piece
<b>Warranty</b>	
European	24 months for manufacturing defects with the exception of consumable parts
<b>Product References</b>	
Manufacturer	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
ISO Code DM 332/99	03.33.06.018
ISO Code DPCM 12.01.2017	04.33.03.015
CND Code	V080701
RDM code	2466418/R