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Materasso ad Aria Level Top Intensive Autodetect.

Destinazione d'uso

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Product Name

Air Mattress Level Top Intensive Autodetect.

Use

The mattress Level Top Intensive Autodetect has to be combined with Devicare R7E pump (code 05.03.R7E). The combination of mattress and pump creates a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

Features	
Code	05.03.TOP-IA
Interchangeable Elements Material	TPU
Number of Elements	18
Cells Height	12 cm
Base	Fireproof Foam Base 5 cm Thick (class 1IM)
Dimensions Inflated	200 x 80 x 17 cm
Therapeutic Capacity	250 Kg
Air Mode	Two Ways Static and Dynamic (A-b-A-b)
AUTODETECT SYSTEM: digital system of automatic weight and patient position	
TRANSPORT connector: allows to keep the air in the mattress for 12 hours without the pump working	
Quick release connectors CPC ONE WAY which allows to change a cell without deflating the mattress	
Straps at the ends for attachment to the bed	
External cover in fireproof Polyurethane with 360° zip on 4 sides	
CPR valve for quick deflation	
First three elements always inflated	
Cover Maintenance and Cleaning	
 Water temperature not above 95° C	 Do not iron
Sterilization is possible in autoclave up to 134° C, rubber cycle	
Mattress Maintenance and Cleaning	
Clean the mattress with a damp cloth and mild soap	
Disinfection is possible with 5% diluted disinfectants even with alcohol base	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
ISO Code DM 332/99	03.33.06.021
ISO Code DPCM 12.01.2017	04.33.06.018
CND Code	V080701
RDM code	2484258/R