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Materasso Ibrido Sadevo Active C.F.A.

**Destinazione d'uso**

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Hybrid Mattress Sadevo Active C.F.A.

**Use**

Device designed for the prevention of bedsores in subjects who have to maintain the lying position for a long time.

| Features   |   |
|--|---|
| Code   | 05.04.H1002CFA  |
| Number of Cells  | 7   |
| Cell Features  | Expandable air cells SADCEL adapt automatically to the shape of the patient according to the weight and the body movement of the latter |
| Dimensions   | 195 x 85 x 16 cm, available in different sizes on request   |
| Therapeutic Capacity   | 160 Kg  |
| The cells are divided in two different compartments, therefore it is possible to connect a pump  |   |
| Two ways STATIC and DYNAMIC mode (A-b-A-b)   |   |
| Totally fireproof, class 1IM   |   |
| Pressure reduction antidecubitus system with pump (Prestige , code 05.03.6600A4 )  |   |
| The OUT-PUT and IN-PUT air flow are regulated by self-adjusting valves placed inside the system  |   |
| AMF viscoelastic foam contact surface (High Density Memory Foam)   |   |
| The external cover in 100% Igni Dahlia (breathable, waterproof, fireproof, two way stretchable and Sanitized fabric) with zip on three sides   |   |
| Ventilated Critical Foot Area (C.F.A.): system allows the heels to sink, reducing the pressure and the friction. This part of the mattress consists of expanding independent cylinders with ultra soft viscoelastic foam. In this way, the risk of creating a decubitus lesion is cleared. |   |
| Cover Maintenance and Cleaning   |   |
|  Water temperature not above 95° C  |  Do not iron   |
| Sterilization is possible in autoclave up to 134° C, rubber cycle  |   |
| Mattress Maintenance and Cleaning  |   |
| Clean the mattress with a damp cloth and mild soap   |   |
| Disinfection is possible with 5% diluted disinfectants even with alcohol base  |   |
| Storage and Disposal   |   |
| Single-user Product  |   |
| Keep away from heat, sparks, open flame or other sources of ignition   |   |
| Dispose of the device in compliance with current regulations on environmental protection and separate waste collection   |   |
| Package and Nr. Items  |   |
| Box  | 1 Piece   |
| Warranty   |   |
| European   | 24 months for manufacturing defects with the exception of consumable parts  |
| Product References   |   |
| Manufacturer   | Demarta Virgino S.r.l.  |
| Directive MDR 2017/745   | Class I   |
| ISO Code DM 332/99   | 03.33.06.018  |
| ISO Code DPCM 12.01.2017   | 04.33.06.018  |
| CND Code   | V080702   |
| RDM code   | 2600011/R   |