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Materasso ad Aria Supercare T02.

**Destinazione d'uso**

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Air Mattress Supercare T02.

## Use

The mattress Supercare T02 has to be combined with a pump create a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

Features	
Code	05.03.T02
Interchangeable Elements Material	PVC 400 Micron
Number of Cells	16
Cells Height	12.5 cm
Dimensions Inflated	200 x 90 x 12.5 cm
Therapeutic Capacity	180 Kg
Air Mode	Two Ways Static and Dynamic (A-b-A-b)
Flaps and straps at the ends for attachment to the bed	
Equipped with PU nylon lining with snap buttons	
Optional: CPR valve for quick deflation and first three elements always inflated	
Cover Maintenance and Cleaning	
 Washing Max 30° C	
Mattress Maintenance and Cleaning	
Clean the mattress with a damp cloth and mild soap	
The mattress has to be dry before use, do not expose it to direct sunlight	
Storage and Disposal	
Single-user product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
ISO Code DM 332/99	03.33.06.021
ISO Code DPCM 12.01.2017	04.33.06.018
CND Code	V080701
RDM code	2466666/R

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Fabbricante Demarta Virgino S.r.l.