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Materasso ad Aria Level Top.

Destinazione d'uso

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Air Mattress Level Top.

Use

The combination of mattress and pump creates a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

Features	
Code	05.03.TOP
Interchangeable Elements Material	Nylon / PU
Number of Cells	18
Cells Height	12 cm
Base	Fireproof Foam 5 cm Thick (class 11M)
Dimensions Inflated	200 x 80 x 17 cm
Therapeutic Capacity	200 Kg
Air Mode	Two Ways Static and Dynamic (A-b-A-b)
TRANSPORT connector: allows to keep the air in the mattress for 12 hours without the pump working	
Straps at the ends for attachment to the bed	
External cover in fireproof Polyurethane with 360° zip on 4 sides	
CPR valve for quick deflation	
First three elements always inflated	
Cover Maintenance and Cleaning	
 Water temperature not above 95° C	 Do not iron
Sterilization is possible in autoclave up to 134° C, rubber cycle	
Mattress Maintenance and Cleaning	
Clean the mattress with a damp cloth and mild soap	
Disinfection is possible with 5% diluted disinfectants even with alcohol base	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virgino S.r.l
Directive MDR 2017/745	Class I
ISO Code DM 332/99	03.33.06.021
ISO Code DPCM 12.01.2017	04.33.06.018
CND Code	V080701
RDM Code	2505130/R