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TWO WHEELS ONE BOTTON FOLDING WALKER WITH ANTI-GLIDE BRAKES
Product Name

Two Wheels (Ø 10 cm) One Botton Folding Walker with anti-glide brakes.

| Code | Length | Width | Height | Weight Capacity | Front Wheel | Net Weight | Carton Size |
|----------------|---|---|---|---|---|---|---|
| Measures in Cm |  |  |  |  |  |  |  |
| 01.06.ZC2E4 | 57.5 | 66 | 82.5/92.5 | 120 | 10 (4") | 3.4 | 55x27x76.5 |

Use

The walkers and rollators are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance.

This aid is designed for a single user, in indoor or outdoor environments according on models.

Description

- Folding and height-adjustable aluminum structure.
- Two front steel swiveling wheels (Ø 10 cm) and two rear anti-slip toes with anti-glide brakes.
- The two lateral sections of the frame are opened and closed by means of the central locking pin.
- Ergonomic and padded knobs.

Materials

| | |
|--------|---------------|
| Frame | Aluminum |
| Knobs | Polypropylene |
| Wheels | PVC and steel |

Maintenance and Cleaning

- Use a damp cloth with mild soap, the water temperature not above 30°C
- Dry well before use
- If necessary, disinfect the product using a common disinfectant detergent
- Do not use water and / or steam jet washing machines
- Must not use acid, alkaline or solvent products such as acetone

Storage and Disposal

- Single-user Product
- Keep away from heat, sparks, open flame or other sources of ignition
- Dispose of the device in compliance with current regulations on environmental protection and separate waste collection

Package and Nr.Items

| | |
|-----|---------|
| Box | 1 Piece |
|-----|---------|

Warranty

| | |
|----------|--|
| European | 24 months for manufacturing defects with the exception of consumable parts |
|----------|--|

Product References

| | |
|------------------------|-------------------------|
| Manufacturer | Demarta Virginio S.r.l. |
| Directive MDR 2017/745 | Class I |
| ISO Code | 12.06.06.003 |
| CND Code | Y120606 |
| RDM Code | 2212422/R |