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Product Name

Low Air Loss Mattress Prime Bariatric.

Use

The Low Air Loss Mattress Prime has to be combined with one pump SD1. The combination creates a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

Features	
Code	05.03.PRIME-120
Interchangeable Elements Material	Nylon / TPU
Number of Elements	20
Cells Height	21 cm
Dimensions Inflated	200 x 120 x 24 cm
Therapeutic Capacity	300 Kg
Air Mode	2 Ways Static and Dynamic (A-B-A-B)
TRANSPORT connector: allows to keep the air in the mattress for 12 hours without the pump working	
17 Elements ventilated	
Foam Base 3 cm height	
Straps at the ends for attachment to the bed	
External cover in fireproof Polyurethane with 2 zips	
CPR valve for quick deflation (<10 sec)	
First three elements always inflated	
SMILE TURN valve that allows 10 ° rotation of the patient on both sides	
Cover Maintenance and Cleaning	
 Water temperature not above 95° C	 Do not iron
Sterilization is possible in autoclave up to 134° C, rubber cycle	
Mattress Maintenance and Cleaning	
Clean the mattress with a damp cloth and mild soap	
Disinfection is possible with 5% diluted disinfectants even with alcohol base	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
ISO Code DM 332/99	03.33.06.021
ISO Code DPCM 12.01.2017	04.33.06.021
CND Code	V080701
RDM Code	2601201/R