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Codice RDM 2601667/R

**Product Name**

Prime Pump for low air loss mattress.

**Use**

The Prime pump has to be combined with low air loss mattress. The combination of mattress and pump creates a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

Features	
Code	05.03.SD1
Dimensions	40 x 15.7 x 27 cm
Case	Fire Retardant ABS
Air Mode	Static and Alternate (A-b-A-b)
Air Output	1000 LPM
Pressure Range	20 to 46 mm Hg
Noise Level	≤ 20 Dba
Cycle Time	5 -10 -15 - 20 - 25 - 30 min
Power	230 V – 50 Hz
Electric Cable Length	4.88 m
Weight	6 Kg
Power Consumpt	25 W
Two steel hooks for anchoring to the bed and air filter	
Visual and sound alarm for low pressure and blackout	
Digital Display	
Nursing Mode: temporary inflating of all elements in static mode	
Pulsing Mode: pressure change from 100% to 70% at intervals of 1 to 20 min	
Automatic detection of sitting position	
SMILE TURN function that allows 10 ° rotation of the patient on both sides	
Warnings	
The pump can be used only with mattresses recommended by the manufacturer	
Check the correct assembly of the device with particular attention to the connection of the tubes between the pump and the mattress	
Maintenance and Cleaning	
Clean the pump with a damp cloth and neutral detergent	
WARNING: never immerse the compressor in any type of liquid	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virgino S.r.l.
Directive MDR 2017/745	Class I
CND Code	V080701
RDM Code	2601667/R

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Fabbricante Demarta Virginio S.r.l.