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Appoggiatesta prono a 2 vie.

Codice	Misure (cm)
05.00.AP811E	23 x 20 x 6

**Destinazione d'uso**

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**Use**

The gel positioners are devices that are used to reach an optimal patient position on the operating table. Many differentes models are available in various shapes and sizes; all fully compatible with human tissues. Some of the main features are: biocompatibility, comfort and extraordinary mechanical resistance.

**Description**

The PRONE HEADREST is made of an amorphous polyurethane gel with a density of 0.99 gr/cm<sup>3</sup>. Thanks to its particular ergonomic design, it can be used as a neck and head support during all types of surgery, with an easy access for intubation.

Materials	
Amorphous Polyurethane Gel with a Density of 0.99 gr/cm <sup>3</sup>	
Maintenance and Cleaning	
The gel positioners can be easily cleaned with any cleaning agent and / or disinfectant for the operating room	
Avoid immersing the gel positioners in the detergent and / or disinfectant for a long time	
Storage and Disposal	
Keep the positioners, if possible, on a level surface	
Avoid placing heavy or sharp objects on the positioners	
Keep the gel temperature between 50 ° and -18 °	
Keep away from sources of light, moisture and heat	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr.Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
ISO Code	04.33.09
CND Code	Y033399
RDM Code	90736/R