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Codice	Misure (cm)
05.00.AP813A	25 x 23 x 6
05.00.AP813B	25 x 18 x 3
05.00.AP813C	45 x 15 x 10

**Destinazione d'uso**

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**Product Name**  
Standard Flat Headrest

Code	Dimensions (cm)
05.00.AP813A	25 x 23 x 6
05.00.AP813B	25 x 18 x 3
05.00.AP813C	45 x 15 x 10

**Use**  
The gel positioners are devices that are used to reach an optimal patient position on the operating table. Many differentes models are available in various shapes and sizes; all fully compatible with human tissues. Some of the main features are: biocompatibility, comfort and extraordinary mechanical resistance.

**Description**  
The STANDARD FLAT HEADREST is made of an amorphous polyurethane gel with a density of 0.99 gr/cm<sup>3</sup>. Thanks to its design, it can be used as a neck and head support during all types of surgery.

<b>Materials</b>	
Amorphous Polyurethane Gel with a Density of 0.99 gr/cm <sup>3</sup>	
<b>Maintenance and Cleaning</b>	
The gel positioners can be easily cleaned with any cleaning agent and / or disinfectant for the operating room	
Avoid immersing the gel positioners in the detergent and / or disinfectant for a long time	
<b>Storage and Disposal</b>	
Keep the positioners, if possible, on a level surface	
Avoid placing heavy or sharp objects on the positioners	
Keep the gel temperature between 50 ° and -18 °	
Keep away from sources of light, moisture and heat	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
<b>Package and Nr.Items</b>	
Box	1 Piece
<b>Warranty</b>	
European	24 months for manufacturing defects with the exception of consumable parts
<b>Product References</b>	
Manufacturer	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
CND Code	Y033399
RDM Code	90736/R