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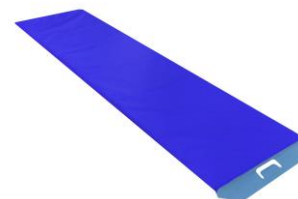
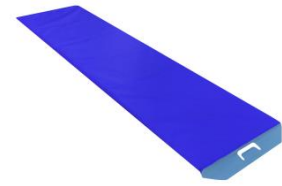
Use

The Transfer Systems are intended for facilitating the manual handling operations of patients through thrust, traction, sliding maneuvers. They have been designed for use in hospitals, nursing homes and healthcare facilities and for home care.

Description

The transfer board SCANGLISS 2 LITE RIGID allows one or more operators to transfer a patient from one surface to another with a difference in height up to 15 cm and in distance up to 20 cm.

The transfer board is equipped with two Padded handles for transport, it is radiolucent. High frequency welding that make the transfer board totally waterproof. It is radiolucent. Maximum Capacity 250 Kg.

Materials	
Handles	Nylon
Base	Etaflon
Base Coating	Nylon
Ultra Low Friction Sheet	Nylon RIP-STOP
Sheet Maintenance and Cleaning	
	Water Temperature not Above 60°C or 140°F
	Do not Iron
	Do not Tumble Dry
	Do not Dryclean
Frame Maintenance and Cleaning	
Disinfection is possible with 5% diluted disinfectants even with alcohol base	
Storage and Disposal	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr.Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
ISO Code	12.31.03.003
CND Code	Y123003
RDM Code	1532910/R