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Rollator Centaurus.

Use

The walkers and rollators are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance. This aid is designed for a single user, in indoor or outdoor environments according on models.

Code	Length	Inside Width	Width	Height	Seat Dimensions	Floor to Seat Height	Weight Capacity	Front Wheel	Rear Wheel	Net Weight	Carton Size
Measures in Cm											
01.06.430	105	50	64	84/97	45x31	55	136	20 (8")	20 (8")	8.9	66x31x47

Features

- Folding aluminum structure with padded seat and fabric back, side stick holder
- Four wheels with a diameter of 20 cm (front swivel wheels with reflector, two fixed rear wheels), there is a lever braking device on the rear wheels
- Removable object-holder bag with handles
- Ergonomic and height-adjustable knobs
- Reversible backrest for double rollator-wheelchair function

Materials

Frame	Aluminum
Wheels	PVC
Seat	PVC

Frame Maintenance and Cleaning

- Use a damp cloth with mild soap
- Do not use water and / or steam jet washing machines, abrasive objects and metal brushes
- Must not use acid, alkaline or solvent products such as acetone

Wheels Maintenance and Cleaning

- Clean the wheels with warm water and neutral detergent
- Lubricate the wheels periodically

Storage and Disposal

- Single-user Product
- Keep away from heat, sparks, open flame or other sources of ignition
- Dispose of the device in compliance with current regulations on environmental protection and separate waste collection

Package and Nr.Items

Box	1 Piece
-----	---------

Warranty

European	24 months for manufacturing defects with the exception of consumable parts
----------	--

Product References

Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
ISO Code	12.06.09.003
CND Code	Y120609
RDM Code	2015903/R