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Deambulatore 2 ruote (Ø 7.5 cm) 2 puntali.

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**Product Name**

Two Wheels (Ø 7.5 cm) One Botton Folding Walker.

| Code           | Length  | Width   | Height  | Weight Capacity   | Front Wheel   | Net Weight  | Carton Size   |
|----------------|---|---|---|---|---|---|---|
| Measures in Cm |  |  |  |  |  |  |  |
| 01.06.ZC1A3F   | 57.5  | 66  | 82.5/92.5   | 120   | 7.5 (3")  | 3.4   | 56x16x69  |

**Use**

The walkers and rollators are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance.

This aid is designed for a single user, in indoor or outdoor environments according on models.

**Description**

- Folding and height-adjustable aluminum structure.
- Two front swiveling wheels (Ø 7.5 cm) and two rear anti-slip toes.
- The front wheels have a directional **locking system**, by operating a lever you can switch from fixed wheels (lever upwards) to swivel wheels (lever downwards).
- The two lateral sections of the frame are opened and closed by means of the central locking pin.
- Ergonomic and padded knobs.

| Materials  |  |
|--|--|
| Frame  | Aluminum   |
| Knobs  | Polypropylene  |
| Wheels   | PVC  |
| Maintenance and Cleaning   |  |
| Use a damp cloth with mild soap, the water temperature not above 30°C  |  |
| Dry well before use  |  |
| If necessary, disinfect the product using a common disinfectant detergent  |  |
| Do not use water and / or steam jet washing machines   |  |
| Must not use acid, alkaline or solvent products such as acetone  |  |
| Storage and Disposal   |  |
| Single-user Product  |  |
| Keep away from heat, sparks, open flame or other sources of ignition   |  |
| Dispose of the device in compliance with current regulations on environmental protection and separate waste collection |  |
| Package and Nr.Items   |  |
| Box  | 1 Piece  |
| Warranty   |  |
| European   | 24 months for manufacturing defects with the exception of consumable parts |
| Product References   |  |
| Manufacturer   | Demarta Virginio S.r.l.  |
| Directive MDR 2017/745   | Class I  |
| ISO Code   | 12.06.06.003   |
| CND Code   | Y120606  |
| RDM Code   | 2212422/R  |