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Deambulatore 4 ruote da 7.5 cm.

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**Product Name**

4 Wheels One Botton Folding Walker (7.5 cm).

| Code           | Length  | Width   | Height  | Weight Capacity   | Front Wheel   | Rear Wheel  | Net Weight  | Carton Size  |
|----------------|---|---|---|---|---|---|---|--|
| Measures in Cm |  |  |  |  |  |  |  |  |
| 01.06.ZD3A3F   | 57.5  | 66  | 82.5/92.5   | 120   | 7.5 (3")  | 7.5 (3")  | 3.4   | 56x16x69   |

**Use**

The walkers and rollators are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance.

This aid is designed for a single user, in indoor or outdoor environments according on models.

**Description**

- Folding and height-adjustable aluminum structure.
- Two **front swivel wheels (Ø 7.5 cm)** with a directional **locking system**, by operating a lever you can switch from fixed wheels (lever upwards) to swivel wheels (lever downwards).
- Two **rear twin wheels (Ø 7.5 cm) with glide brakes**.
- The two lateral sections of the frame are opened and closed by means of the central locking pin.
- Ergonomic and padded knobs.

**Materials**

|        |               |
|--------|---------------|
| Frame  | Aluminum      |
| Knobs  | Polypropylene |
| Wheels | PVC           |

**Maintenance and Cleaning**

Use a damp cloth with mild soap, the water temperature not above 30°C

Dry well before use

If necessary, disinfect the product using a common disinfectant detergent

Do not use water and / or steam jet washing machines

Must not use acid, alkaline or solvent products such as acetone

**Storage and Disposal**

Single-user Product

Keep away from heat, sparks, open flame or other sources of ignition

Dispose of the device in compliance with current regulations on environmental protection and separate waste collection

**Package and Nr.Items**

|     |         |
|-----|---------|
| Box | 1 Piece |
|-----|---------|

**Warranty**

|          |  |
|----------|--|
| European | 24 months for manufacturing defects with the exception of consumable parts |
|----------|--|

**Product References**

|                       |                         |
|-----------------------|-------------------------|
| Manufacturer          | Demarta Virginio S.a.S. |
| Directive UE 2017/745 | Class I                 |
| ISO Code              | 12.06.06.006            |
| CND Code              | Y120606                 |
| RDM Code              | 2212422/R               |