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Product Name

Air Mattress Supercare T01 Bariatric with Prestige pump.

Use

The mattress Supercare T01 Bariatric has to be combined with Prestige pump (05.03.6600A4). The combination of mattress and pump creates a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

Anti-bedsore Kit	
Code	05.03.A4+T01-120
Mattress Features	
Interchangeable Elements Material	Nylon / PVC
Number of Cells	18
Cells Height	12 cm
Dimensions Inflated	200 x 120 x 12 cm
Therapeutic Capacity	200 Kg
Air Mode	Two Ways Static and Dynamic (A-b-A-b)
Flaps and straps at the ends for attachment to the bed	
External cover in fireproof Nylon with press studs	
CPR valve for quick deflation	
First three elements always inflated	
Cover Maintenance and Cleaning	
🧺 Washing Max 30° C	
Mattress Maintenance and Cleaning	
Clean the mattress with a damp cloth and mild soap	
The mattress has to be dry before use, do not expose it to direct sunlight	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Features Pump	
Dimensions	24 x 11.5 x 9 cm
Case	Fire Retardant ABS
Air Mode	Alternate (A-b-A-b)
Air Output	7 LPM
Pressure Range	50 to 120 mm Hg
Noise Level	≤20 Dba
Cycle Time	12 min
Power	220/240 V-50 Hz
Power Consumption	7 Watt
Electric Cable Length	1.7 m
Weight	0.8 Kg
Fuse	1.5 A

Other Features	Two steel hooks for anchoring to the bed
Warnings	
The pump can be used only with mattresses recommended by the manufacturer	
Check the correct assembly of the device with particular attention to the connection of the tubes between the pump and the mattress	
Maintenance and Cleaning	
Clean the pump with a damp cloth and neutral detergent	
WARNING: never immerse the compressor in any type of liquid	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
ISO Code DM 332/99	03.33.06.021
ISO Code DPCM 12.01.2017	04.33.06.018
CND Code	V080701
RDM code	2466662/R