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Use

Polymoove Positioners are a valid aid for healthcare professionals for patient positioning. They are designed to give comfort, support, stability and protection of vulnerable areas. Used as a support they adapt to the morphology of the body excellently.

Description

The POLYMOOVE SQUARE PLATE POSITIONER is made of latest generation EvoPore foam. This thermosensitive foam has high elasticity, stability and offers optimal support in patient positioning.
Can be used as: pillow, trunk support, upper and lower limb support.
The use of the EvoPore plate with positioners for kinesiology practices is recommended.
The fire retardant, breathable, bi-elastic, waterproof and antibacterial OBATEX lining is made of polyurethane and polyester, sterilizable in an autoclave.

Positioning Kit:

- POLYMOOVE Rectangular Positioner 15x40 cm (ref. 03.05.POL10)
- POLYMOOVE Squadre plate in EvoPore 50x50x3 cm (ref. 03.05. R4018)
- POLYMOOVE Rectangular Positioner 50x75 cm (ref. 03.05.POL32)
- POLYMOOVE Rectangular Positioner 55x95 cm (ref. 03.05.POL34)
- POLYMOOVE Linear Dorsal Positioner 220x36 cm (ref. 03.05.POL91)

Materials	
Foam Sheet	EvoPore Foam
External Cover	Obatex
External Cover Maintenance and Cleaning	
	Water temperature not above 95°C or 200°F
	Do not bleach
	Do not iron
	Dryclean with any solvent
	Tumble dry Max temperature 60°C or 140°F
Sterilization is possible in autoclave up to 134° C, rubber cycle	
Mattress Maintenance and Cleaning	
Possible cleaning with a damp cloth and neutral detergent	
Before use dry the mattress carefully without exposing it to direct sunlight	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr.Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	

Manufacturer	OBA AG
Authorized Distributor	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
CND Code	Y060399
RDM Code	225942/R